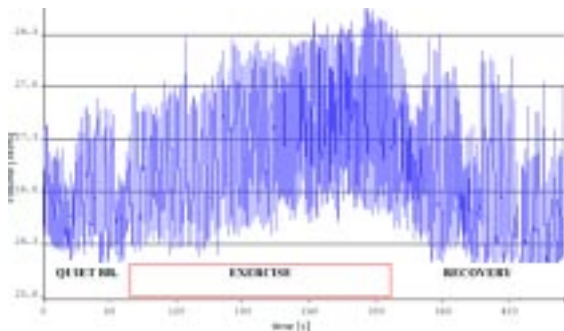
BTS OEP SYSTEM

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**BTS OEP System:
a new tool for the
breath-by-breath monitoring
of dynamic hyperinflation
during exercise in COPD
patients**

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
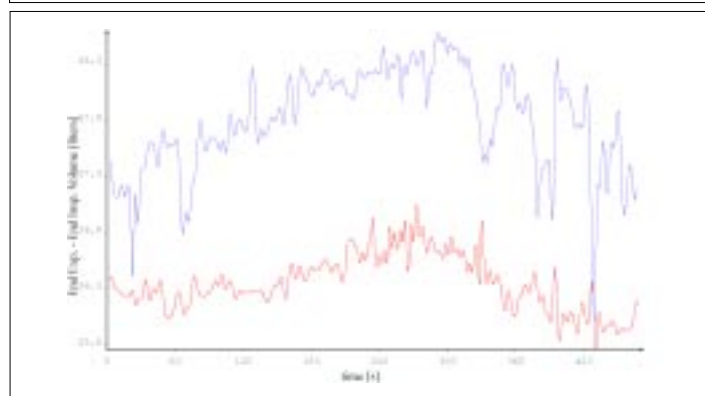
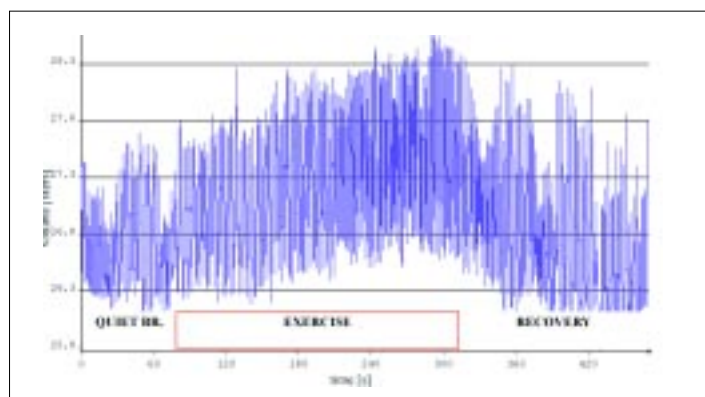
1

2

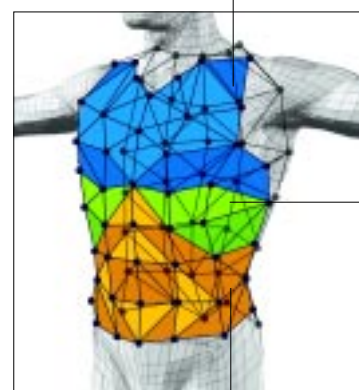BTS OEP System: a new tool for the breath-by-breath monitoring of dynamic hyperinflation during exercise in COPD patients

In patients with Chronic Obstructive Pulmonary Disease (COPD), progressive dynamic hyperinflation leads to intolerable sensations of breathlessness that make an important contribution to exercise limitation.

In these patients changes in end-expiratory lung volume (EELV) therefore, constitute a fundamental parameter for functional evaluation and an important outcome for assessing the effects of therapeutic interventions during exercise. However, up to now it was impossible to follow EELV during exercise. Flowmeters cannot be used for long periods of time, mainly because integration of flow at the mouth suffers from measurement drift, so that changes in absolute lung volume are not accurately recorded.

Changes in EELV are therefore routinely carried out by serial inspiratory capacity (IC) maneuvers, but this procedure has three major important limitations:

- the patient must be able to cooperate (i.e. to perform a number of perfectly executed maneuvers during exercise);
- inability to follow breath-to-breath variations in lung volume;
- limitations in identifying the contribution of the different chest wall compartments.

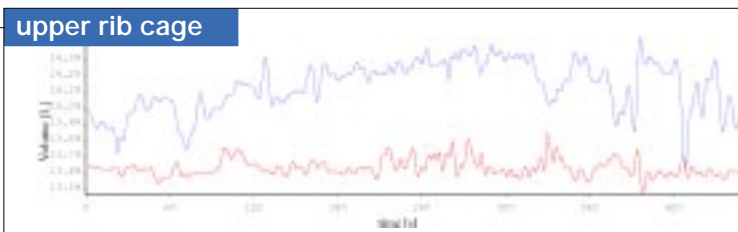
Today these problems can be overcome through the application of OEP technology. The BTS OEP System measures operating lung volume on a breath-by-breath basis (1).

Results are obtained through the measurement of chest wall volume (V_{cw}). It is evident that if breath-by-breath variations in end-expiratory and end-inspiratory lung volume can be monitored by BTS OEP System, the complete ventilatory pattern, including tidal volume, respiratory rate, duty cycle, inspiratory and expiratory times, mean inspiratory flow, can also be measured.


In addition, BTS OEP System can measure breath-by-breath variations in end-inspiratory and end-expiratory V_{cw} and volume variations of the different chest wall compartments (2).

BTS OEP System is therefore the state-of-the-art for the complete monitor of ventilation in COPD patients during exercise, as well as during any other conditions (e.g. spontaneous quiet breathing, invasive and noninvasive mechanical ventilation,...)

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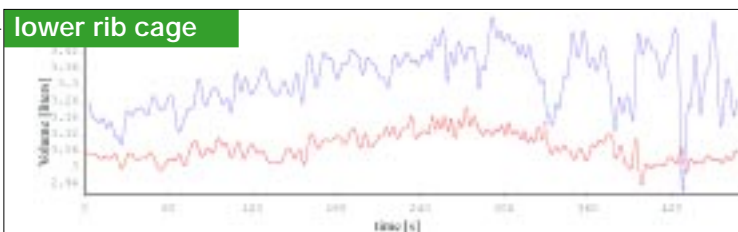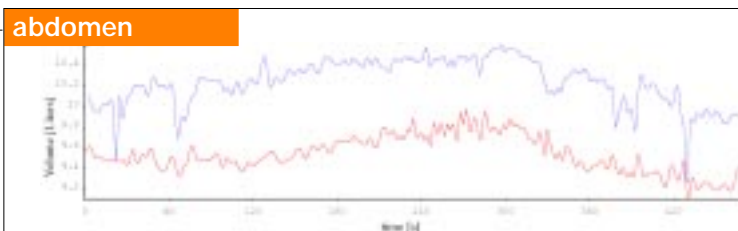
OEP System and COPD: selected bibliography

Recent studies, based on OEP System measurements, have shown that the time course of EELV changes, how the volume change is distributed between the compartments of the chest wall (rib cage and abdomen), and what happens in patients who do not show these changes in lung volume are crucial for understanding the pathophysiology of COPD patients during exercise.

Aliverti A, Rodger K, Dellaca RL, Stevenson N, Lo Mauro A, Pedotti A, Calverley PM.

The effect of salbutamol on lung function and chest wall volumes at rest and during exercise in COPD.

Thorax. 2005 doi:10.1136/thx.2004.037937

Vogiatzis I, Georgiadou O, Golemati S, Aliverti A, Kosmas E, Kastanakis E, Geladas N, Koutsoukou A, Nanas S, Zakyntinos S, Roussos C.

Patterns of dynamic hyperinflation during exercise and recovery in patients with severe chronic obstructive pulmonary disease.

Thorax. 2005;60(9):723-9.

Calverley PM, Koulouris NG.

Flow limitation and dynamic hyperinflation: key concept in modern respiratory physiology.

Eur Respir. J2005;25:186-199.

Aliverti A, Stevenson N, Dellaca RL, Lo Mauro A, Pedotti A, Calverley PM.

Regional chest wall volumes during exercise in chronic obstructive pulmonary disease.

Thorax. 2004;59(3):210-6.

De Groote A, Van Muylem A, Scillia P, Cheron G, Verleden G, Paiva M, Estenne M.

Ventilation asymmetry after transplantation for emphysema: role of chest wall and mediastinum.

Am J Respir Crit Care Med. 2004;170(11):1233-8.

Bianchi R, Gigliotti F, Romagnoli I, Lanini B, Castellani C, Grazzini M, Scano G.

Chest wall kinematics and breathlessness during pursed-lip breathing in patients with COPD.

Chest. 2004;125(2):459-65.

Duranti R, Filippelli M, Bianchi R, Romagnoli I, Pellegrino R, Brusasco V, Scano G.

Inspiratory capacity and decrease in lung hyperinflation with albuterol in COPD.

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Estimation of end-expiratory lung volume variations by optoelectronic plethysmography.

Crit Care Med. 2001; 29(9):1807-11.

Aliverti A, Macklem PT.

How and why exercise is impaired in COPD.

Respiration. 2001;68(3):229-39.

BTS Spa

viale Forlanini 40
20024 Garbagnate Milanese (MI)
Italy
Tel. +39 02 99514 549
Fax +39 02 99514 599
www.bts.it

BTS Bioengineering